

STEP BY STEP

1



CAPTURE YOUR TRANSFORMATION BY TAKING A SELFIE BEFORE STARTING TO COMPARE WITH THE RESULTS AFTERWARDS.

2



USE THE MATCH WHITE SHADE GUIDE TO CHECK YOUR SHADE BEFORE STARTING THE WHITENING SESSION.

3



POP THE SCRUB ONTO THE PALM OF YOUR HAND AND THEN PLACE IT ONTO THE TOOTHBRUSH THEN CLEAN YOUR TEETH GENTLY. DO NOT OVER-BRUSH AND AVOID YOUR GUMS. SPIT INTO A SPIT BAG AND DO NOT RINSE AFTERWARDS.

4



EVENLY SPREAD UP TO THE **1ST USE** MARK OF THE WHITENING GEL ONTO BOTH THE UPPER AND LOWER PARTS OF THE MATCH WHITE MOUTH GUARD. ENSURE THE GEL IS EVENLY DISTRIBUTED ON THE SECTIONS THAT WILL BE IN CONTACT WITH YOUR FRONT TEETH. REST THE MOUTH GUARD ON THE PROVIDED BEAUTY ROLL.

5



PLACE THE TOOTH WIPE ON YOUR INDEX FINGER AND SPRAY IT 2-3 TIMES WITH THE PREP SPRAY. AVOID SPRAYING DIRECTLY INTO YOUR MOUTH.

6



WIPE OVER YOUR TEETH WITH THE HARLEY STREET SMILES TEETH WIPE, COVERING ALL OF OUR SMILE AREA, AVOIDING THE GUMS. RESIST THE URGE TO CLOSE YOUR MOUTH AT THIS STAGE AS THIS MAY DILUTE THE PRODUCT AND IMPACT ON THE RESULTS.

7



INSERT THE GEL-FILLED MATCH WHITE MOUTH GUARD INTO YOUR MOUTH AND BITE DOWN GENTLY. MOVE YOUR JAW FROM SIDE TO SIDE. CHECK IN THE MIRROR THAT YOU HAVE EVEN DISTRIBUTION OF THE GEL ON YOUR SMILE AREA. WEAR YOUR PROTECTIVE GOGGLES. USE THE REMOTE TO ACTIVATE THE LASER LIGHT. THE LIGHT WILL AUTOMATICALLY TURN OFF AFTER 20 MINUTES.

8



AFTER THE LIGHT TURNS OFF, DISCARD THE CONTENTS INTO THE SPIT BAG OR BEAUTY ROLL PROVIDED. DRY WIPE THE MOUTH GUARD WITH A TISSUE TO REMOVE ALL GEL, THEN REPEAT STEPS 4 TO 7 TWO MORE TIMES TO COMPLETE A TOTAL OF THREE 15 MINUTE CYCLES. CHECK YOUR RESULTS USING SHADE GUIDE. SHARE YOUR NEW SMILE! HASHTAG #MATCHWHITEOFFICIAL AND TAG @MATCHWHITEOFFICIAL FOR A CHANCE OF WINNING A PRODUCT FROM OUR RETAIL RANGE!